



I-DROP[®] PUR GEL



DE
FI
IT
PL
SV

**PRESERVATIVE-FREE
INSTANT, LONG LASTING HYDRATION
BLINK ACTIVATED
0.30% Viscoadaptive hyaluronan**

SUPERIOR RELIEF



**SANS AGENT DE CONSERVATION
HYDRATATION INSTANTANÉE ET DE LONGUE
DURÉE | ACTIVÉES PAR LE CLIGNEMENT
Hyaluronane viscoadaptif 0,30 %**

SOULAGEMENT SUPÉRIEUR



INDICATIONS

For prolonged relief of discomfort due to minor irritations of the eye from exposure to wind, sun, water or other irritants. For use as a lubricant and protectant against further irritation or to relieve dryness of the eye. I-DROP[®] PUR GEL is intended to be used within 12 months of opening. **Do not use beyond expiry date** ().

DIRECTIONS

Wash hands before use. Do not use after the expiry date. Instill 1 drop in the affected eye(s) as needed. While looking in a mirror, gently pull down lower eyelid and instill 1 drop on its inner surface or apply 1 drop directly to the corneal surface of the eye by leaning back to a horizontal position. After application, shake any remaining droplet from tip of bottle and replace cap securely. Store in a cool, clean dry place.

TIPS FOR USE

1. Invert the bottle and squeeze it gently until a drop forms on the tip. This may take a couple of seconds as the solution needs to travel through the one-way-valve system.
2. With the drop suspended at the tip, carefully bring the bottle above your eye and squeeze gently until it falls into the affected eye.
3. Please note: After use, there may be a residual amount of liquid remaining on the blue tip. This confirms that the bottle is working as it was designed, and not drawing this liquid back into the bottle.

If any residual solution remains on the bottle after use, simply flick it off with a brisk motion and close the cap tightly.

This step is critical in keeping the remaining solution sterile and safe. Even a minuscule amount of fluid, exposed to a non-sterile external environment, may become contaminated.

COMPOSITION

I-DROP[®] PUR GEL is a sterile, preservative-free, blink-activated viscoadaptive lubricating eye drop and contains: water for injection, sodium chloride, sodium hyaluronate, glycerin, sodium phosphate.

WARNINGS

If you experience eye pain, change in vision, continued redness or irritation of the eye, or if the condition worsens or persists for more than 72 hours, discontinue use and consult a doctor.

I-DROP[®] PUR GEL may modify the effect of other solutions or drugs that you may be using on the eye. This product has not been tested during pregnancy or lactation. Do not apply I-DROP[®] PUR GEL on an infected eye.

Do not operate machinery or drive a vehicle immediately after use. To avoid contamination, do not touch tip of container to any surface. Do not use this product if you are allergic to any of the ingredients. Replace cap after use. Keep bottle tightly capped. Not for use with contact lenses. Do not use if bottle appears damaged or compromised.

KEEP OUT OF REACH OF CHILDREN.

HOW IT'S SUPPLIED

I-DROP[®] PUR GEL is supplied in a bottle with a fill volume of 10 mL. Do not use if the packaging is damaged.



STORE BETWEEN 2°C AND 25°C.
DO NOT USE BEYOND EXPIRY DATE ().

Developed by Biocia Inc.
I-DROP[®] is a registered trademark of I-MED Pharma Inc. ©2022 I-MED Pharma Inc.



ENTREPOSER ENTRE 2 °C ET 25 °C.
NE PAS UTILISER AU-DELÀ DE LA DATE DE PÉREMPTION ().

Conçu par Biocia Inc.
I-DROP[®] est une marque déposée de I-MED Pharma Inc. ©2022 I-MED Pharma Inc.

Manufactured in France for:
Fabriqué en France pour :
I-MED PHARMA INC.
7190 Frederick Banting Rd.
St. Laurent, QC
Canada H4S 2A1
(514) 685-8118
info@imedpharma.com

EC REP
ARAZY GROUP GmbH The Square 12
Am Flughafen, 60549 Frankfurt
am Main Germany
UK RESPONSIBLE PERSON: MEDES LIMITED
5 Beaumont Gate, Shenley Hill, Radlett,
Hertfordshire WD7 7AR, England, UK
medes@arazygroup.com

CH REP
MedEnvoy Switzerland
Gothardsstrasse 28
6302 Zug
Switzerland

MedEnvoy
Prinses Margrietplantsoen 33 - Suite 123
2595 AM The Hague
The Netherlands

REF: 205
I-DPGELPI 0222

imedpharma.com

