**PRESERVATIVE-FREE  
INSTANT, LONG LASTING HYDRATION  
BLINK ACTIVATED**  
0.30% Viscoadaptive hyaluronan

**SUPERIOR RELIEF**

**INTENDED PURPOSE**

The device is intended to provide prolonged relief of symptoms and discomfort for patients suffering from evaporative or aqueous deficient dry eye disease (DED). The devices may also be used by otherwise healthy persons to provide temporary relief of discomfort due to minor irritation of the eye from exposure to wind, sun, water and other irritants.

**INDICATIONS**

Severe chronic dry eye disease.

**DIRECTIONS**

Wash hands before use. Instill one drop in the affected eye(s) as needed. While looking in a mirror, gently pull down lower eyelid and instill one drop on its inner surface or apply one drop directly to the corneal surface of the eye by leaning back to a horizontal position.

Please note, after use, there may be a residual amount of liquid remaining on the blue tip. This confirms that the bottle is working as it was designed, and not drawing this liquid back into the bottle. If any residual solution remains on the bottle after use, simply flick it off with a brisk motion and close the cap tightly.

This step is critical in keeping the exterior surface of the bottle clean and dry. Even a minuscule amount of fluid, exposed to a non-sterile external environment, may become contaminated and thus contaminate the exterior of the bottle.

Store in a cool, clean dry place. I-DROP® PUR GEL is intended to be used within 12 months of opening.

**CONTRA-INDICATIONS**

Do not use if you are allergic to any of the ingredients.

**INTENDED PATIENT POPULATION**

The target treatment population is patients suffering from moderate to severe dry eye disease. Not to be used by children under three years of age.

**TIPS FOR USE**

1. In order to facilitate application of the eye drop, invert the bottle and squeeze it gently until a drop forms on the tip. This may take a few seconds as the solution needs to travel through the one-way-valve system.
2. With the drop suspended at the tip of the inverted bottle, carefully bring the bottle above your eye and squeeze gently until it falls into the affected eye.

**COMPOSITION**

I-DROP® PUR GEL is a sterile, preservative-free, blink-activated viscoadaptive lubricating eye drop and contains: water for injection, sodium chloride, sodium hyaluronate, glycerin, sodium phosphate.

**WARNINGS**

If you experience eye pain, change in vision, continued redness or irritation of the eye, or if the condition worsens or persists for more than 72 hours, discontinue use and consult a doctor. Serious incidents should be reported to the manufacturer and local competent authority.

If you are using any medication administered to the eye, please consult your eye care professional prior to using I-DROP® PUR GEL. This product has not been tested during pregnancy or lactation. Do not apply I-DROP® PUR GEL on an infected eye.

Due to the expected temporary blurring after application (due to the high viscosity of our I-DROP®s), do not operate machinery or drive a vehicle immediately after use. To avoid contamination, do not touch tip of container to any surface. Do not use this product if you are allergic to any of the ingredients. Replace cap tightly after each use. Do not use if bottle or packaging appears damaged or compromised. Do not use after the expiry date.

**KEEP OUT OF REACH OF CHILDREN.**

**HOW IT'S SUPPLIED**

I-DROP® PUR GEL is supplied in a bottle with a fill volume of 10 mL. Ensure used bottles are disposed of in plastic recycling bins. Box and insert can be disposed of in paper recycling bins.

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STORE BETWEEN 2°C AND 25°C.  
DO NOT USE BEYOND EXPIRY DATE (☞).

I-DROP® is a registered trademark of I-MED Pharma Inc. ©2023 I-MED Pharma Inc.

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**UK REP**  
**MEDES LIMITED**  
5 Beaumont Gate, Shenley Hill, Radlett,  
Hertfordshire WD7 7AR,  
England, UK

**CH REP**  
**MedEnvoy Switzerland**  
Gothardstrasse 28  
6302 Zug  
Switzerland

**Grafton Optical Company Ltd**  
Unit 7, Riverpark Industrial Estate,  
Billet Ln, Berkhamsted HP4 1HL,  
United Kingdom

**MedEnvoy**  
Prinses Margrietplantsoen 33 - Suite 123  
2595 AM The Hague  
The Netherlands

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I-DROP® on I-MED Pharma Inc:n rekisteröity tavaramerkki. ©2023 I-MED Pharma Inc.
I-DROP® è un marchio registrato di I-MED Pharma Inc. © 2023 I-MED Pharma Inc.