EN

HOT & COLD THERAPY EYE MASK WITH THERMABEADS™

Soft, reusable, eye mask provides moist heat therapy for the relief of symptoms associated with Blepharitis, Dry Eye Disease, Meibomian Gland Disease (MGD), Styes and Chalazia.

CONTAINS

- 1 Eye mask 8.6 in x 3.5 in (22cm x 9 cm)
- 1 Protective slipcover

INDICATIONS

Hot & Cold Therapy Eye Mask with ThermoBeads™ for the relief of symptoms associated with:

- Blepharitis
- Dry Eye Disease
- MGD
- Styes
- Chalazia
- Headaches
- Sinus pressure
- Tension
- Swelling
- Puffiness

DIRECTIONS

Use protective slipcover to keep mask clean of oils, makeup and dirt. Remove slipcover and wash as required.

HOT THERAPY

Use hot therapy for relief of symptoms associated with Blepharitis, Dry Eye Disease, Meibomian Gland Disease (MGD), Styes and Chalazia.

DIRECTIONS

1. I-RELIEF™ mask must be at room temperature before heating.
2. Insert I-RELIEF™ mask inside protective slipcover (optional) and place flat on clean microwave safe dish.
3. **SET MICROWAVE POWER TO HIGH AND HEAT FOR 20 SECONDS. Before applying, always touch the I-RELIEF™ mask with fingers first, to test for desired temperature. If the mask is too hot, allow it to cool for 1-2 minutes and test again. For additional heat, microwave in 5 second increments until desired temperature is reached. Do not exceed a maximum of 30 seconds of heating.**
4. Place I-RELIEF™ mask over closed eyes and secure with adjustable strap, if preferred.
5. Wear for 3 to 5 minutes or as directed by a physician. **Do not apply for more than 20 minutes at a time.**

COLD THERAPY

Use cold therapy to help relieve headaches, sinus pressure, tension, swelling and puffiness.

DIRECTIONS

1. Insert I-RELIEF™ mask inside protective slipcover (optional) and place in a sealable plastic bag, in the freezer for 30 minutes.
2. Remove I-RELIEF™ mask from freezer and plastic bag, and place over closed eyes, securing with adjustable strap, if preferred. Wear for 5 to 7 minutes or as directed by a physician.

I-RELIEF™ EYE MASK CARE

Remove slipcover. Hand wash only in cold water with a mild detergent. Rinse thoroughly. Gently squeeze out excess water (do not twist). Lay flat on clean, ventilated surface for a minimum of 24 hours or until fully dry. Wash immediately if exposed to food, oils, grease or other dirt.

SLIPCOVER CARE

Machine wash in warm water on regular cycle with like colors, using regular detergent. Machine dry on low heat. Wash slipcover as often as necessary.

STORAGE

Store I-RELIEF™ mask in cool dry place. Keep away from flames.

WARNINGS AND PRECAUTIONS

Please read all information before use.

- **For external use only.**
- **Keep out of reach of children.**
- Do not apply for more than 20 minutes at a time.
- Dispose of I-RELIEF™ mask if punctured or broken.
- Always test I-RELIEF™ mask temperature with fingers prior to use on eyes.
- Overheating or prolonged application may cause injury.
- Do not use on infants.
- Do not apply to sensitive skin, skin with no sensation, or open wounds.
- Do not boil.
- Overheating in a microwave may damage I-RELIEF™ mask.
- Do not ingest contents
- Warm according to instructions. Do not apply if too hot.
- Do not fall asleep using I-RELIEF™ mask.
- People with diabetes, circulation disorders, skin infections, etc. should consult with their doctor before use.
- Consult your doctor if your condition worsens or irritation develops
- Serious incidents should be reported to the manufacturer and local competent authority.

The I-RELIEF™ mask can be disposed of as normal waste in the garbage.

The I-RELIEF™ box and insert can be disposed of in paper recycling bins.

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I-RELIEF™ is a trademark of I-MED Pharma Inc. ©2022 I-MED Pharma Inc.
ARAZY GROUP GmbH
Grafton Optical Company Ltd